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Intended use and indication

Our rubber tips are used to maintain, promote and secure walking in conjunction with a walking aid or walking stick. **Please also be sure to read the safety instructions!**

Contraindication

Not available.

General safety information

Before each use, check your walking aid for proper functioning and a tight fit of the rubber tip. Check the rubber tip regularly for damage or wear and replace it if necessary. The grip of the underground is crucial for the safe hold of the walking aids! Avoid wet floors or surfaces such as gravel, sand, leaves, ice or snow! Pay attention to stumbling points like unevenness, holes in the ground, loose floor coverings or edges, because the going assistance can slip away unexpectedly or sink in! Special caution is appropriate with insufficient lighting conditions, since then irregularities in the underground are only badly recognizable!

Mounting the rubber capsule

Remove the old rubber tip from your walking aid or walking stick and put the new rubber tip on the tube. **Attention!** Be sure to check the tight fit of the rubber tip.

Use

The risk of slipping or falling- especially on slippery and wet surfaces- can never be completely ruled out when using walking aids. When walking, place the crutch or walking stick as vertically as possible and not too far away from the body. A very oblique placement increases the risk of falling!

Storage and care

Never store the rubber tip near heat sources such as ovens, heaters or in the car on the shelf! Do not expose the rubber tip to sunlight for a longer period of time! Strong sun or heat will damage the material! **The rubber tip is a wearing part and must be replaced regularly. Warning!** Never clean rubber tips with oily substances! Danger of slipping! If necessary, clean the rubber tip with a mild cleaning agent and a soft cloth if it is dirty. **Never** use cleaners containing solvents, as these will damage the material! **Note:** If disinfection becomes necessary, use a suitable disinfectant. Please note the manufacturer's instructions for use.

Limitation of use

Damaged and heavily worn rubber tips must not be used under any circumstances, as they no longer provide the required safety! **Note:** All serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Re-use (rental) / duration of use

We accept liability in accordance with the Product Liability Act only for the initial placing of our products on the market. We only agree to a reuse if the product has been tested by us beforehand. **The service life of a rubber tip depends on wear, but is limited to a maximum of 2 years.** Beyond this time, further use of the rubber tip is the responsibility of the user.

Warranty and liability


You have purchased a high-quality Ossenberg product. If, despite the highest quality standards, a justified material defect should occur, Ossenberg has the right, within the framework of a 12-month warranty, to repair the defective item or to replace it with a defect-free subsequent delivery, at its own discretion. We cannot accept any liability for damage caused by modifications to our products. Not covered by the warranty are defects that are due to wear or improper handling. Wear parts are excluded from the warranty. Only original spare parts and accessories may be used! For spare parts, accessories and repairs, please contact the specialist trade.

Disposal

Please consult your specialist dealer about the regional disposal regulations.

Technical data

REF		A
716	Rubber tip with steel inlay	16 mm
719	Rubber tip with steel inlay	19 mm

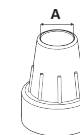Symbol directory

Item number	Batch number	Read the operating instructions	This product complies with the requirements of Regulation (EU) 2017/745 (MDR).
Manufacture	Date of manufacture	Attention	Medical device
Distribution by	Permissible body weight		

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